














# Set-up Guide



## 1. Connect

- Connect the power cable, and ethernet cable (if available)

## 2. Set preferences

- Set operating language: scroll , then  to confirm
- If no ethernet is detected:
  -  to scan for Wifi networks, scroll  to select, then  to confirm
  - Input password: Use number keys, or scroll   to select characters. Move cursor using  
  -  to save password, then   to continue
- Note 16-digit Lapbox ID, then  to exit set-up

## 3. Register Lapbox

For new clients: Create your account at [mastatestsystem.com](http://mastatestsystem.com)





For existing customers:


- Login to your [mastatestsystem.com](http://mastatestsystem.com) account
- Select "My Account" and "Register a Lapbox" from menu

## Caring for your Mastatest system


- Recommended room temperature of 5-30°C (41-86°F) for best performance
- Position Lapbox on a clean, dry desk space and handle with care
- Keep internal and external surfaces clean using a damp cloth (water only)
- Store cartridges at 2-8°C (35-46°F), and use before expiry date

## User Information

-  Red light indicates power on.
-  Green light indicates internet connection.
-  to return to previous screen.
-  long-press to delete a character entry.

Set-up menu options are always available by pressing  when on main testing screen


## Fill a Mastatest® cartridge

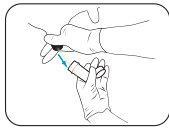


Take a clean milk sample (see user videos)




Put on clean gloves, and remove cartridge from pack

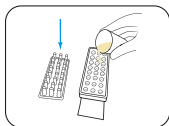


Remove cartridge lid and half-fill cartridge with milk

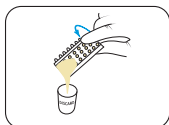


Pour off surplus milk and discard

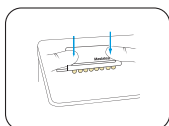

Replace lid, pushing down firmly, and label lid with cow details

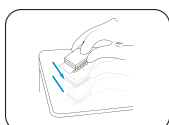



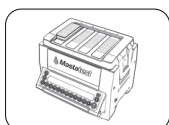

Tap firmly 2-3 times, repeating if air-bubbles are visible in wells






Insert cartridge into Lapbox™ with arrows facing to the back and replace lid

## Start a Mastatest




1. Start position defaults to first available.  to confirm, or scroll   to chosen position, then 









Cartridge type should be automatically detected.

If not select Single (1 test)   
or Twin (2 tests) cartridge  

2. Select **BoxStart** , or **WebStart**  

3. Enter animal details:

- BoxStart: Enter animal number eg: **1** **2** **3**   
and scroll to select quarter  
- WebStart: Enter at [mastatestsystem.com](http://mastatestsystem.com)

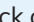- When test has started, a countdown timer shows on screen. Further tests can be started at any time.
-  to return to prior screen.
- Long-press  to delete.
- To check or cancel a test in progress, scroll to select position   , then  to check or   to cancel.



For more information [mastatest.com/user-guides](http://mastatest.com/user-guides)  
or [support@mastatest.com](mailto:support@mastatest.com)

## CONTACT US

### Producers

1240 Green Valley Road  
Beaver Dam, WI 53916  
800.255.1181 Fax: 920.885.2812  
[info@armorah.com](mailto:info@armorah.com)

844 Bennie Road  
Cortland, NY 13045  
800.767.5611 Fax: 920.885.2812  
[info@armorah.com](mailto:info@armorah.com)

1440 Action Drive SE  
Mandan, ND 58554  
888.930.9378 Fax: 701.663.9638  
[beeforders@armorah.com](mailto:beeforders@armorah.com)

27058 Mueller Place Suite 1  
Sioux Falls, SD 57108  
800.255.1181  
[info@armorah.com](mailto:info@armorah.com)

